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Descriptif

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
Walking aid

Description

This belt can be used on the patient or on the caregiver.

- When the belt is positioned on the caregiver, the patient can grip the handles to secure themselves and position themselves in an antepulsion position to facilitate transfer.
- When the belt is positioned on the patient, it allows the carer to secure them but also to encourage a rocking motion to facilitate walking. The carer can position themselves behind or in front of the patient.
- Waist size: S: 75/90 cm, M: 85/105 cm, L: 105/120, XL: 120/135

CADM2-PF

v 01.07/25

Recommendation

The use of the device requires the presence and vigilance of a third person.
This medical device must be implemented or trained by a health professional.

This belt can be used on the patient or on the caregiver.

- When the belt is positioned on the caregiver, the patient can grip the handles to secure themselves and position themselves in an antepulsion position to facilitate transfer.
- When the belt is positioned on the patient, it allows the carer to secure them but also to encourage a rocking motion to facilitate walking. The carer can position themselves behind or in front of the patient.
- Waist size: S: 75/90 cm, M: 85/105 cm, L: 105/120, XL: 120/135

Composition

- Combination of polyurethan foam and jersey/cotton polyester fabric
- Jersey/PU coating
- Polyester strap
- 100% polyester fabric

Set up

This medical device is intended for patient positioning. This is not a restraint system.
Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.