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Descriptif

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Sliding bed sheets with handles

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
Description

The sliding sheet with handles makes it easier for carers to grip and move the patient, thereby reducing the risk of musculoskeletal disorders.

Its use prevents shearing.

Available in two widths: 70 and 110 cm.

Available in several lengths: width 70: 80, 130 and 195 cm / width 110 cm: 140 cm.

DTR-PF

v 07.02/25

Recommendation

Use of the device requires the presence and vigilance of a third party. This medical device must be used or trained by a healthcare professional.

The sliding sheet with handles makes it easier for carers to grip and move the patient, thereby reducing the risk of musculoskeletal disorders.

Its use prevents shearing.

Composition

- 100% polyamide warp and weft
- Silicone coating on 1 side
- 100% polyester handles

Description

Made from a polyester tube of synthetic fabric (polyamide with silicone-treated polyurethane coating) allowing the two sides of fabric in contact to slide over each other.

The addition of handles and the sliding effect facilitate the work of care staff by reducing friction due to the weight of the patient during handling.

Characteristics

- Our sheets are waterproof.
- Can be disinfected with disinfectant solutions such as surfanios, microdefender, etc.
- Machine washable at a maximum temperature of 70°C.
- Can be tumble-dried at a maximum temperature of 100°C.

Video demonstration

Before each use, it is important to check the presence of the label, the stitches of the straps, that they do not fray or break, that the device does not show any tears, snags, holes or signs of fraying (hanging fabric and straps)

SCAN ME