## Drap de glisse

### Descriptif

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
## Sliding bed sheet

### Description

For patient handling and repositioning, the sliding sheet is a device that facilitates the transfer of a patient lying in bed, thereby reducing the risk of back pain and musculoskeletal disorders for caregivers.

Available in 3 widths: 48, 70 and 110 cm. Available in several lengths: width 70: 80, 130 and 195 cm / width 110: 140 cm / width 48: 48 cm.

DGL-PF

v 07.02/25

## Recommendation

Use of the device requires the presence and vigilance of a third party.  
This medical device must be used or trained by a healthcare professional.

For patient handling and repositioning, the sliding sheet is a device that facilitates the transfer of a patient lying in bed, thereby reducing the risk of back pain and musculoskeletal disorders for caregivers.

## Composition

- 100% polyamide warp and weft
- 1-sided silicone coating

## Description

It consists of a polyester tube of synthetic fabric (polyamide with silicone-treated polyurethane coating) allowing the two sides of fabric in contact to slide over each other.

The sliding effect facilitates the work of nursing staff by reducing friction due to the weight of the patient during handling.

## Characteristics

- Our sheets are waterproof.
- Can be disinfected with disinfectant solutions such as surfanios, microdefender, etc.
- Machine washable at a maximum temperature of 70°C.
- Can be tumble-dried at a maximum temperature of 100°C.

## Video demonstration

Before each use, it is important to check that: the label is present, and the device has no tears, tears, holes or signs of fraying.

SCAN ME