Protections de barrières de lit

Descriptif

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
FOR ALL BEDS

Description

The foam protectors are easy to install on bed rails, preventing patients from bumping into or getting their limbs caught between the bars, while also protecting them from direct contact with the metal. Thanks to their new ergonomic shape, they stay in place on their own while you reach for the 25 mm quick-release fasteners, making them easy to install. The removable cover, made of M1-rated Protectlaize, complies with fire safety standards. Available in size M (140 cm de longueur) and L (180 cm de longueur). Sold individually.

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CE

Recommendation

The use of the device requires the presence and vigilance of a third person. This medical device must be implemented or trained by a health professional.

The siderail pads are installed on the bed barriers and prevent the patient from bumping and moving its limbs through the bed rails, thus protecting him from contact with the metal. The quick and easy fixations to the siderails is made thanks to adjustable buckles (25 mm). The removable foam cover is manufactured in a flame retardant fabric (classified CRIB 5).

Composition

- Complete fabric composition: 77% PVC 23% PES
- Polyester sling and nylon buckle
- Polyurethane foam

Set up

This medical device is intended for patient safety.

Before each use, it is important to check the presence of the label, the stitches of the straps, that they do not fray or break, that the device does not show any tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

- Place the cotsides bumpers inside the side rails, zipper down, against the mattress.
- Close the buckles and adjust tight the straps around the tubes of the side rails.
- Check if the cotsides bumpers are well fixed to the side rails, without any possibility to turn around.

Important recommendations

- In order to get a maximum efficiency of these sort of product, it is necessary :
 - to choose the appropriate size depending on the bed
 - to get the best possible adjustment on the bed
- These products must not be in direct contact with a wounded skin.

Cleaning :

- Clean with a soft cloth and warm soapy water on the surface of the cotsides bumper.
- Desinfect the surface of the cotside bumper with any used desinfection product.

Cleaning of the covers :

- Never wash the covers together with the foam.