NAUSICAA

MEDICAL

www.nausicaa-medical.com

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This device is intended lifts and ceiling-mounted lifts for transferring a person to a sitting position.

It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

UNIVERSAL ECO TOILET SLING

Compatible with
boned headrest
(TABAL).

The Universal Eco Toilet Sling is recommended for toilet and daily change transfers.

Its extended U-shape completely frees the patient's buttocks thus facilitating the positioning to the toilet, and improving the installation and removal of the sling.

Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.

STORP3-PF-TU

The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.

This medical device must be implemented or trained by a health professional.

V07.07/26

Important Recommendations :


- In order to get a maximum efficiency of these sort of product, it is necessary to get the best possible adjustment to the patient.
- These products must not be in direct contact with a wounded skin.

UNIVERSAL ECO TOILET SLING

SIZE & DIMENSIONS (in cm)

TU	A	78
	B	25
	C	107

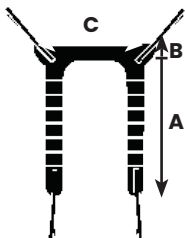Composition
Jersey 100 % polyester
Sangle polyester 40 mm - 1450 DaN
Mousse PE 50 kg/m³

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

The backrest of this type of strap, is constituted by the wide part of the strap.

The thigh supports, meanwhile, are constituted by the branches of the U.

This type of strap has a hanging system to the flail of the lift in four points:

1 at each end of the thigh supports.

Position the bottom of the backrest at the level of the shoulder blades so that the ears of the system of attachment are on both sides of the person. Pass the fasteners of the back splash under the arms of the person, so that the bead is under the armpits.

While slightly lifting the right leg of the person, pass from the outside of the thigh inward the attachment of the right thigh support, proceed similarly for the support of left thigh. Hook the two backrest fasteners and the two thigh support clips to the flail of the lift.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

End-of-life management

- Medical devices at the end of their useful life (obsolete, beyond repair or past their service life) must not be disposed of with household waste.
- The user is responsible for decommissioning them, cleaning and disinfecting them, sorting the materials and directing them to the appropriate treatment channels, in accordance with current regulations.
- Textile components (polyester, polyamide, polyurethane, nylon) and plastics must be recycled where possible.
- Used or damaged slings must no longer be used for lifting.
- Contaminated devices must be disinfected and then disposed of in accordance with applicable procedures, in particular the DASRI regulations for infectious waste.
- Material recovery and recycling must be prioritised to ensure the sustainable, safe and compliant management of medical devices at the end of their life.