www.nausicaa-medical.com

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www.nausicaa-medical.com

This device is intended lifts and ceiling-mounted lifts for transferring a person to a sitting position.

It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

QUICK TOILET SLING

Compatible with
removable headrest
(TABAL).

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
Its enlarged U-shape completely clears the patient's buttocks, making it easier to position the patient on the toilet and to put the sling on and take it off.

This sling has a narrow headband and no back foam. The legs pass from the outside to the inside. This sling prevents constriction of the hips when used uncrossed.

Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.

STO-PF

The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.

This medical device must be implemented or trained by a health professional.

V08.07/26

Important Recommendations :


- In order to get a maximum efficiency of these sort of product, it is necessary :
 - to choose the appropriate size for the patient
 - to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.

QUICK TOILET SLING

SIZE & DIMENSIONS (in cm)

S	A	67
	B	26
	C	78
M	A	85
	B	26
	C	95
L	A	90
	B	26
	C	108
XL	A	100
	B	26
	C	122

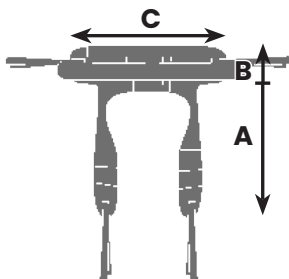Composition
100% Polyester Fabric
50Kg/M3 PE foam
100% Polyester
20Kg/M3 35mm PE foam

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This type of sling has a 4 points hanging system to the spreader bar :

- 2 on both sides of the back
- 1 at each end of the thigh support.

Position the lowest part of the back on the shoulder blades, so that the hanging loops are positioned on both sides of the patient. Place the hanging loops under the patient's arm, so that the foam parts are positioned under the armpits. Close the chest loop.

Slightly raise the right leg of the patient, pass from the inside of the thighs outwards the loop of the right thigh support, do the same for the left thigh support. Hang both back loops and both thigh supports to the spreader bar.

ADJUSTMENTS :

The different loops on the hanging straps allow various sitting positions.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

HEADREST INSTALLATION (OPTIONAL)

The removable, whalebone headrest TABAL-PF is available as an option on the eco toilet sling models, and is installed as follows:

TABAL-PF: The removable, whalebone headrest slides between the sling and the patient, protecting the cervical vertebrae when picking up from the floor, for example...

End-of-life management