www.nausicaa-medical.com

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www.nausicaa-medical.com

This device is intended lifts and ceiling-mounted lifts for transferring a person to a sitting position.

It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

BACKREST TOILET SLING

Compatible with
removable headrest
(TABAL).

CE certified and made in France, the Fast Transfer Strap is recommended for transfers to toilets and exchange.

Its widened U-shape completely clears the patient's buttocks thus facilitating positioning in the toilet, and improves the positioning and removal of the strap.

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
The passage of the legs is from the inside to the outside of where its indented form.

This strap avoids the constriction of the hips.

This strap is not recommended for patients with very low tone.

Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.

SATL-PF

The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.

This medical device must be implemented or trained by a health professional.

V07.07/26

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.


BACKREST TOILET SLING

SIZES & DIMENSIONS (in cm)

	Waist size	A	B	C
S	70/90	71	23	81
M	90/115	86	23	95
L	115/140	98	23	109
XL	140/165	108	23	123

Composition

100% polyester jersey
Polyester webbing 40 mm - 1450 DaN
PE foam 50 kg / m3
Polyurethane foam 20 kg / m3
40 mm polyester straps
Auto gripping 100% Polyester

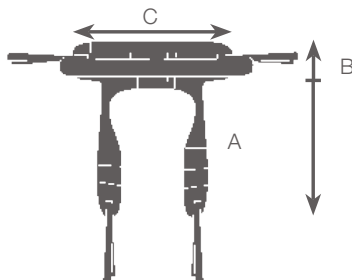The backrest of this type of strap, is constituted by the wide part of the strap.

The thigh supports, meanwhile, are constituted by the branches of the U.

This type of strap has a system of attachment to the flail of the lift in four points:

1 at each end of the thigh supports.

Position the bottom of the backrest at the level of the shoulder blades so that the ears of the system of attachment are on both sides of the person. Pass the fasteners of the backsplash under the arms of the person, so that the beads are under the armpits. Close the plastron.

While slightly lifting the right leg of the person, pass from the outside of the thigh inward the attachment of the right thigh support, proceed similarly for the support of left thigh. Hook the two backrest fasteners and the two thigh support clips to the flail of the lift.

2, on both sides of the backsplash

SETTINGS

The attachment system, by multiple ears, allows to vary the sitting position of the person.

CAUTION

Read the instructions for use before using the straps.

Backsplash lines must be hung on the same color.

Never change the setting of the hooking ears on the hooks of the mobile lift when a person is in the strap.

Before straightening a person with the mobile lift, make sure that the strap is hooked securely to the hooks.

End-of-life management