

SLINGS

NAUSICAA
MEDICAL

Made in France
since 1992




Universal eco sling

Description

Its U-shape avoids the placement of the sling under the patient's buttocks, making the installation and removal comfortable for the patient and accommodating for the caregiver.
One size.

TU	A	71
	B	67
	C	91

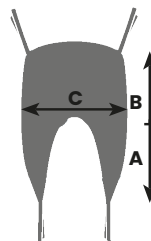

SUP3G-PF-TU / SUP3-PF-TU / SURP3-PF-TU

V02.07/26

Recommendation



The use of the device requires the presence and vigilance of a third person. This medical device must be implemented or trained by a health professional.

Composition

- 100% polyester fabric
- 40mm polyester strap - 1450 DaN
- 20mm polyester bias

Comfort model:
• 50kg/m³ PE foam

Set up

This device is intended lifts for transferring a person to a sitting position. It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This type of sling has a 4 points hanging system to the spreader bar :

- on both sides of the back
- at each end of the thigh support.

Position the lower part of the sling at the coccyx so that the loops of the hanging straps are on both sides of the patient. Slightly raise the right leg of the patient, pass the right thigh support under the right thigh. Do the same for the left leg. Both hanging straps are positioned between the legs of the patient. Cross these hanging straps. Hang both hanging straps of the back and hanging straps of the thigh supports to the spreader bar.



Important recommendations

- In order to get a maximum efficiency of these sort of product, it is necessary to get the best possible adjustment to the patient.
- These products must not be in direct contact with a wounded skin.

Caution

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

Installing the headrest (optional)

The TAMOV-PF removable headrest is available as an option on eco toilet sling models. It is installed as follows:

TAMOV-PF: The removable headrest slides between the sling and the patient, protecting the cervical spine when lifting from the floor, for example.



End-of-life management

