SANGLES

NAUSICAA
MEDICAL

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

Enveloping sling

Description

The integrated boned headrest and the enveloping shape of this strap are recommended for transferring agitated and anxious patients. Its U-shape prevents the strap from being placed under the patient's buttocks, making it more comfortable for the patient and easier for the caregiver to put on and take off. The positioning of the hooks at shoulder level prevents compression of the patient during transfer.

Sizes

	S	M	L	XL
A	60	73	74	77
B	85	90	94	102
C	97	110	124	136

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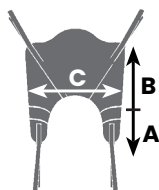V07.07/26

Recommendation

The use of the device requires the presence and vigilance of a third person. This medical device must be implemented or trained by a health professional.

Composition

- Tissu 100% polyester ou vinyle armé
- Sangle polyester 40 mm 1450 DaN
- Mousse PE 50 kg/m³
- Baleinage aluminium 15x3 mm
- Biais polyester 40 mm

Set up

This device is intended lifts and ceiling-mounted lifts for transferring a person to a sitting position. It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments. Remove the aluminium bar before putting the sling in the washing machine.

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This type of sling has a 4 points hanging system to the spreader bar:

- 2 on both sides of the back
- 1 at each end of the thigh support.

Position the lower part of the sling at the coccyx so that the loops of the hanging straps are on both sides of the patient. Slightly raise the right leg of the patient, pass the right thigh support under the right thigh. Do the same for the left leg. Both hanging straps are positioned between the legs of the patient. Cross these hanging straps. Hang both hanging straps of the back and hanging straps of the thigh supports to the spreader bar.

ADJUSTMENTS: The differents loops on the hanging straps allow various sitting positions.

Important recommendations

- In order to get a maximum efficiency of these sort of product, it is necessary :
 - to choose the appropriate size for the patient
 - to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.

Caution

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

End-of-life management