NAUSICAA

MEDICAL

www.nausicaa-medical.com

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This device is intended lifts and ceiling-mounted lifts for transferring a person to a sitting position.

It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

UNIVERSAL ECO HAMMOCK SLING

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
Compatible with
boned headrest
(TAMOV).

- CE certified and manufactured in France, the Universal Eco Hammock is recommended for “lifting from the floor” transfers in a lying position.
- Its simple and indented cut on the sides makes it the preferred sling to let it under the patient in a chair as it shows no thickness and folds which cause decubitus ulcers and discomfort.
- The installation must be done in bed in a lying position at 180° by practicing a lateralization.
- Moreover, it is preferred for the bath or shower, because it is made with a high tenacity polyester net in order to evacuate the water.

Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.

HA-PF-TU / HAG-PF-TU / HA3-RBA-PF-TU

The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.

This medical device must be implemented or trained by a health professional.

V02.07/26

Important Recommendations :


- In order to get a maximum efficiency of these sort of product, it is necessary :
 - to choose the appropriate size for the patient
 - to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.

UNIVERSAL ECO HAMMOCK SLING

SIZE & DIMENSIONS (in cm)

TU	A	74
	B	121

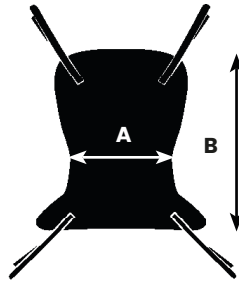Composition
100% polyester fabric
40mm polyester strap - 1450 DaN
40mm polyester bias sling

Model with handle

- Lateralize the patient. Fold the hammock in two along the vertical axis.
- Place the fold along the vertical axis at the center of the patient's back.
- Place half of the sling on the free side of the patient.
- Fold the side of the sling on the side in support of the patient.
- Turn the patient on his back with half the sling resting on the free side.
- Lateralize again the patient on the previously free side of the sling.
- Release the previously folded part and spread it out.
- Place the patient on his back on the sling deployed.
- Hang the high and low hangers on the patient lift's spreader bar.

CAUTION

Read the instruction manual before using the slings.

The backrest hangers must be hung on the same color.

Never change the attachment setting of the ears to the stand-up lift hooks, when someone is installed in the sling.

Before lifting a person with the stand-up lift, make sure the sling is securely hooked to the hooks.

End-of-life management