

## SLINGS

NAUSICAA  
MEDICAL

Made in France  
since 1992



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
# Standard hammock sling

## Description

Its simple cut and low-cut sides make it the ideal strap to leave under the patient in the chair, as it does not have any excess thickness or folds that can cause pressure sores and discomfort. The comfort foam under the legs reduces shear. It should be positioned in bed in a 180° supine position with lateralisation.

## Recommendation

The use of the device requires the presence and vigilance of a third person. This medical device must be implemented or trained by a health professional.



## Composition

- 100% polyester jersey / 100% polyester mesh
- 40 mm polyester strap - 1450 DaN
- 50 kg/m3 PE foam (Comfort model)
- 40 mm polyester bias binding




SHA2-PF / SHAC2-PF / SHA2BA-PF / SHAC2BA-PF

V07.06/26

## Sizes

|   | S  | M   | L   | XL  |
|---|----|-----|-----|-----|
| A | 62 | 66  | 74  | 83  |
| B | 96 | 104 | 113 | 121 |

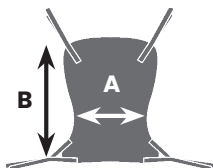

## Set up

This device is intended lifts and ceiling-mounted lifts for transferring a person to a sitting position. It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.

- Turn the patient onto their side. Fold the hammock in half along the vertical axis.
- Position the fold along the vertical axis in the middle of the patient's back.
- Place half of the strap on the patient's free side.
- Fold the side of the strap on the patient's support side.
- Turn the patient onto their back with half of the strap placed on the free side.
- Turn the patient back onto their side on the strap on the previously free side.
- Unfold the previously folded part and spread it out.
- Place the patient on their back on the unfolded strap.
- Attach the upper and lower slings to the middle of the LP.



## ADJUSTMENTS

The multiple hanging straps allow various positions : sitting, semi-sitting or lying position.

## Important recommendations

- In order to get a maximum efficiency of these sort of product, it is necessary :
  - to choose the appropriate size for the patient
  - to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.

## Caution

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level. Do not change any adjustment while the patient is hanging in the sling. Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

## Installation of headrest (option)

The TAMOV-PF headrest with fasteners and the TAAT-PF removable headrest are available as options on hammock strap models. They are installed as follows:

TAMOV-PF: The removable headrest slides between the strap and the patient.

TAAT-PF: Pass the fabric straps sewn onto the straps through the loops on the headrest.

Check that the straps are taut, then attach the headrest suspension straps to the spreader bar so that the patient can rest their head comfortably.



## End-of-life management

