# NAUSICAA

---

## MEDICAL

[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

### BLUE WAYUP 3

FR

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ENGLISH

# Contents

## User manual / Stand up lift : BLUE WAYUP 3

|                                            |       |
|--------------------------------------------|-------|
| • Use, contraindications, content          | 33    |
| • Installation instructions                | 34-35 |
| • Use of the device                        | 36-39 |
| • Using the motorization                   | 40    |
| • Using the sling                          | 41-42 |
| • Storage                                  | 43-44 |
| • Re-use of the stand up lift / labelling  | 45    |
| • Labelling                                | 46-47 |
| • Preventive maintenance and safety checks | 48-51 |
| • General safety instructions              | 52    |
| • Technical and dimensional specifications | 53    |
| • Spare Parts                              | 54    |
| • Cleaning and maintenance                 | 55    |
| • Technical specifications motorization    | 56-57 |
| • Troubleshooting guide                    | 58    |
| • Guarantee                                | 59    |

**BLUE WAYUP 3**

**NAUSICAA**  
MEDICAL

# Presentation

The BLUE WAYUP 3 sit-to-stand lift is a Class I medical device in accordance with Regulation (EU) 2017/745. All our devices comply with the NF en ISO 10535 : 2021 standard.

It can be used in hospitals, clinics, retirement homes, EHPAD, at home... and must be handled by a caregiver. It is designed for use with an appropriate strap.

This device is designed for patients weighing up to 150 kg.

Climatic conditions for use of the BLUE WAYUP 3 are as follows: ambient temperature from 0°C to 40°C, humidity from 20% to 80% and air pressure from 700 to 1060 hPa.

The device can be used in the bathroom (without water contact) or toilet.

The lift enables the patient to be actively transferred from a bed, recliner, toilet or wheelchair.

ENGLISH

## Contraindications

Conditions such as osteogenesis imperfecta, osteoporosis, spinal disease, mental illness or epilepsy may be contraindications.

## Content

The items listed in the following table are included with the unit :

| Box          | Designation                          | Quantity |
|--------------|--------------------------------------|----------|
| BLUE WAYUP 3 | Stand up lift                        | 1        |
|              | Control unit with integrated battery | 1        |
|              | Removable battery                    | 1        |
|              | Remote control                       | 1        |
|              | Knee support                         | 1        |

# Installation instruction

BEFORE USING YOUR LIFT, PLEASE CHECK THAT :

- Wheels turn and roll normally.
- The rear wheel brakes are working properly.
- There is no wear or deformation on the hooks.

## INSTALLATION

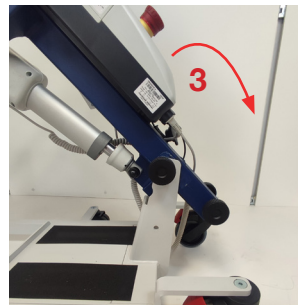
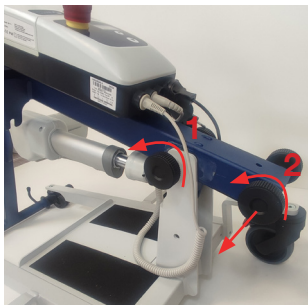
The unit is delivered folded in half with its knee support removed.

**Step 1:** Loosen the top knobs to release the mast **without removing them.**

**Step 2:** Completely unscrew the lower knobs and **remove the mast axle.**

**Step 3:** Raise the upper part of the unit **by rotating it.**


# Installation instruction

**Step 4:** Replace the axle, washers and knurls in the bottom hole.

Step 5: Firmly tighten the four knurls to secure the assembly.

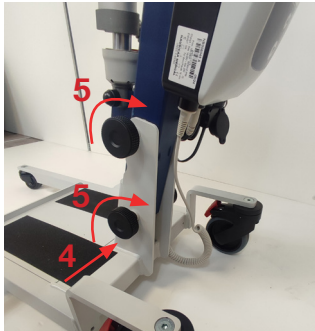**Step 6:** Insert the knee support into its housing and tighten the knob to secure it.


BEFORE USING YOUR LIFT, PLEASE CHECK THAT :

- Wheels turn and roll correctly.
- Brakes function correctly.
- There is no wear or deformation on the hooks.
- The various components are correctly connected.
- The battery is sufficiently charged.

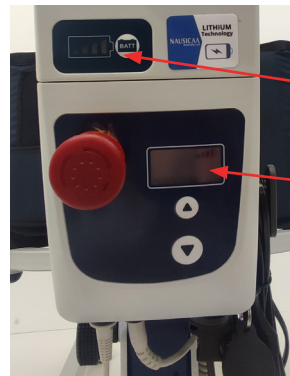Cylinder connector  
Remote control  
connector

Battery level display

LCD display

# Use of the device

## INSTRUCTIONS FOR USE:

- Your sit-to-stand lift is designed for transferring a patient, and must not be used for any other purpose or for moving a patient over long distances.
  - Check that the patient's weight does not exceed the maximum weight the lift can support.
  - Operate the lift by pushing the handlebars, never by pushing the patient.
  - The lift must be handled with care when transferring a patient, and at a speed appropriate to the situation.
  - Use the lift on flat, smooth surfaces. It is not recommended to use the lift on a slope of more than 5°. If you have to use a ramp, it is advisable to have a second person help you. Never use a sit-to-stand lift under a running shower.
- Never recharge the batteries of a sit-to-stand lift near a bath or shower.
- Sit-to-stand lifts are useful and effective when a patient has a certain degree of dependence and needs a caregiver for basic care.
  - They contribute to patient stimulation and mobility; stimulate the cardiac system; combat osteoporosis and disorders associated with immobility, such as falls during transfers; stimulate brain activity, helping to maintain urinary and fecal continence; improve bowel activity and bladder function.

ATTENTION :  
DO NOT USE  
THE APPLIANCE WITHOUT  
THE KNEE SUPPORT

# Use of the device

- In order to use a sit-to-stand lift, it is necessary to assess the patient's physical capabilities.
- When using a sit-to-stand lift, patients can be separated into two categories:
  1. Patients with low muscle tone
  2. Patients with average muscle tone
- How to use a sit-to-stand lift :
- Patients with low muscle tone should be transferred to a semi-seated position.

- Patients with average muscle tone should be transferred to a standing position.
- For safe and effective raising, the patient should place his or her hands on the handlebars

(see photo below).

- In addition, the patient's shins must be in contact with the knee-rest at all times.
- Positioning of the knee support must be carried out by a caregiver.

- Positioning of the knee support must be carried out by a caregiver
- Depending on the patient's height, adjust the position of the knee support strap by sliding it on

# Use of the device

## Braking:

Brakes are an important safety feature of the lift. The rear wheels are equipped with brakes that act on the wheels. To lock the wheels, press the red part of the wheel with your foot until it stops (braked wheel). To release, push the red part of the wheel upwards (unbraked wheel).

When locking wheels, always ensure that both wheels are locked.

If only one wheel is locked on sloping ground, the unlocked wheel will roll away. (The lift may tip sideways).

## FIXE MODEL

## ECP MECHANICAL AND ELECTRICAL MODEL

Small  
front wheel

Large  
rear wheel

Unbrake  
front wheel

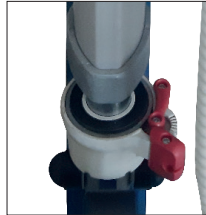
Braked  
rear wheel

# Use of the device


Before use, check that the remote control and actuator are correctly connected to the control unit (page 22).  
Check the condition of the control unit and battery.

Check that the cylinder is correctly secured.

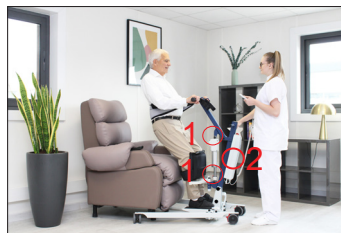Check that the battery is charged using the display on the battery. Press and hold the BATT button to display the charge level.


# Using the Motorisation

Remote control for fixe model and ECP mechanical

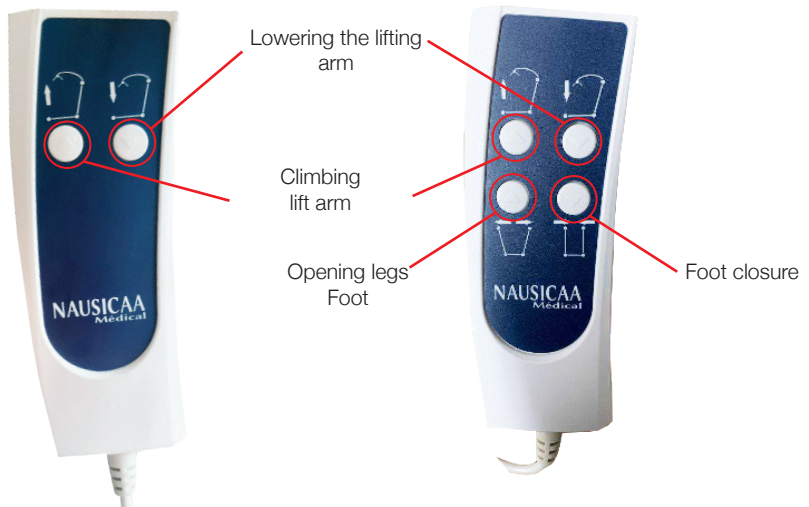Remote control for fixe model and ECP mechanical


# Using the sling

Nausicaa Médical recommends using our PREMIUM and VERTICONFORT back straps. Nausicaa Médical lift straps are all compatible with Nausicaa Médical lift systems. For advice on choosing the right product for your needs, please contact our sales department. To select a strap size, the caregiver must match the chest circumference (sub-axillary strap) or waist circumference (back strap) with the dimensions shown in the documentation.

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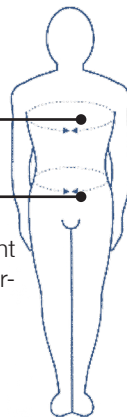|            |       |        |         |         |
|------------|-------|--------|---------|---------|
| Sizes      | S     | M      | L       | XL      |
| Chest size | 70/90 | 90/115 | 115/140 | 140/165 |

Before using a sit-to-stand lift, the following parameters need to be taken into account depending on the situation: the patient's physical handicap, pathology and general morphology. Our R&D department has introduced a color code to quickly identify the size of each strap:

- size S is yellow
- size M is red
- size L is green
- size XL is blue
- size TU (One Size Only) is black

| Poids / Taille | 1m50 | 1m55 | 1m60 | 1m65 | 1m70 | 1m75 | 1m80 | 1m85 | 1m90 | + 1m90 |
|----------------|------|------|------|------|------|------|------|------|------|--------|
| 40 kg          | S    | S    | S    | S    | S    | S    | M    | M    | M    | M      |
| 45 kg          | S    | S    | S    | S    | S    | S    | M    | M    | M    | M      |
| 50 kg          | S    | S    | S    | S    | S    | S    | M    | M    | M    | M      |
| 55 kg          | S    | S    | S    | S    | S    | S    | M    | M    | M    | M      |
| 60 kg          | M    | M    | M    | S    | S    | S    | M    | M    | M    | M      |
| 65 kg          | M    | M    | M    | S    | S    | S    | M    | M    | M    | M      |
| 70 kg          | M    | M    | M    | M    | M    | M    | M    | M    | M    | M      |
| 75 kg          | L    | L    | M    | M    | M    | M    | M    | M    | M    | M      |
| 80 kg          | L    | L    | L    | L    | M    | M    | M    | M    | M    | M      |
| 85 kg          | L    | L    | L    | L    | M    | M    | M    | M    | M    | M      |
| 90 kg          | L    | L    | L    | L    | L    | L    | M    | M    | M    | M      |
| 95 kg          | XL   | XL   | XL   | XL   | L    | L    | M    | M    | M    | M      |
| 100 kg         | XL   | XL   | XL   | XL   | L    | L    | L    | L    | L    | L      |
| 110 kg         | XL   | XL   | XL   | XL   | L    | L    | L    | L    | L    | L      |
| 120 kg         | XL   | XL   | XL   | XL   | XL   | XL   | L    | L    | L    | L      |
| 130 kg         | XL   | XL   | XL   | XL   | XL   | XL   | XL   | XL   | XL   | XL     |
| 160 kg         | 2 XL | 2 XL | 2 XL | 2 XL | 2 XL | 2 XL | XL   | XL   | XL   | XL     |

### IMPORTANT RECOMMENDATIONS

- TO ENSURE MAXIMUM EFFICIENCY WHEN USING THIS RANGE OF PRODUCTS

PRODUCTS IN THIS RANGE, IT IS ESSENTIAL TO :

- CHOOSE THE RIGHT SIZE FOR THE PATIENT

- FIT THE PRODUCT AS SNUGLY AS POSSIBLE ON THE PATIENT

- THESE PRODUCTS MUST NOT BE PLACED IN DIRECT CONTACT WITH DAMAGED SKIN.

- READ THE INSTRUCTIONS FOR USE BEFORE USING THE STRAPS, AND CARRY OUT THE ABOVE CHECKS BEFORE EACH USE.

- BEFORE TRANSFERRING A PATIENT WITH THE SIT-TO-STAND LIFT, MAKE SURE THE SLING IS SECURELY ATTACHED TO BOTH HOOKS ON THE LIFT.

- VEST LIFE: 3 YEARS

# Using the sling

Position the bottom of the backrest at lumbar level, so that the strap hooks are accessible from either side of the patient.

Pass the backrest straps under the patient's arms. Attach the two ties to the hooks on the sit-to-stand lift.

## ADJUSTMENTS

The multi-hanger system allows you to adjust the person's position.

## PATIENT VERTICALIZATION

The strap is positioned at waist level, and the grip adjustment is made with the strap under tension and the patient's arms positioned at 120°. The hands are placed on the handles and the arms are stretched to activate chest tone.

Fast, full extension of the lifting arm gives a standing position. The partial extension of the lift arm, combined with the buttock strap, creates a sitting position.

These straps help to secure the patient during deambulation.

## ISO 10535: 2021 STANDARD ON INSPECTION :

### B.2.4 Checking body support elements (= straps)

The soft support element should be checked periodically at the frequency specified by the manufacturer, at least twice a year.

More frequent checks may be necessary when a soft support element is used or cleaned more frequently than normal.

Inspections should be carried out by a suitably qualified person who is familiar with the model of support element and its use and maintenance.

The inspection should enable signs of deterioration, wear or potential failure to be detected and to check the legibility of the labels.

The inspection report should be kept in a secure place, so that it can be consulted in the event of an incident.

This record should include the following information:

- date of inspection;
- identification details of the support element and its serial number;
- information on the condition of the support element;
- the next scheduled inspection date;
- the inspector's n

# Storage

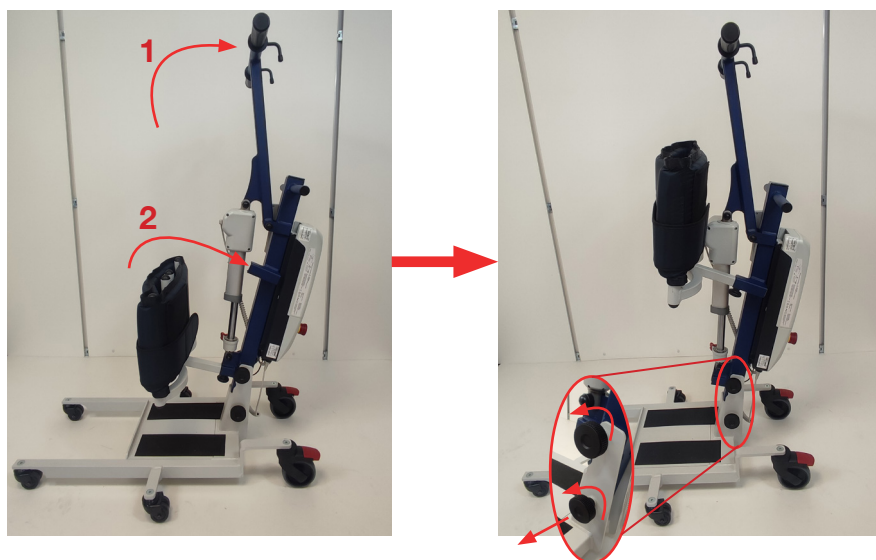
To optimize storage, the BLUE WAYUP 3 foldable is designed to be folded for storage.

To do this, follow the steps described below:

**Step 1:** Use the remote control to raise the arm to the top position.

**Step 2:** Unscrew the knee support and insert it into the top bracket. Tighten the knob after inserting the knee support.


**Step 3:** Loosen the knobs at the top of the base without removing them. Unscrew the knobs at the bottom of the baseplate and remove the axle, holding the upper part of the lift to prevent it from falling forward.

# Storage

**Step 4:** Tilt the top section forward to fold the lift in half, then screw the axle washers and knurls at the base of the mast.

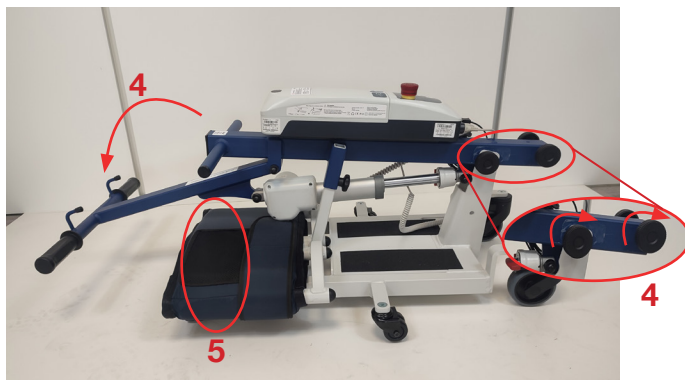**Step 5 :** Pass the knee support strap under the chassis to keep it folded.

- The lift can now be raised using the arm as a handle.
- It can be wheeled like a trolley or caddy to its storage location.
- The unit is designed to be stored in an upright position (an optional cover is available).

Dimensions L x l x H = 590 x 490 x 1100

# Re use of the sit-to-stand lift

The lift can be re-used.

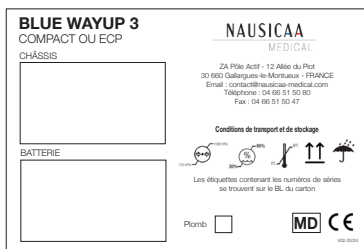
The lift and sling must be cleaned, disinfected and maintained after each use.

Please refer to chapters :

- Preventive Maintenance & Safety Checks
- Cleaning & Care

## Labelling

Labelling on the box

# Labelling

1

**SAFETY INSTRUCTIONS - Before each use**

**NAUSICAA**

MEDICAL

- Check the perfect condition of the hoist sling.
- The sling must never be used to move the hoist.
- Check the perfect condition of the traction sling (Easylev range).
- Verify the presence and conditions of safety devices : all axis and their constrained fixations (pins, clips and bolts) as well as the tightness of the screws (see user manuals).

V01.03/23

**DO NOT USE THE DEVICE IN CASE OF ANY LACK OR BAD OVERALL STATE**  
NAUSICAA Médical - 12 Allée du Piot - ZA Pôle Actif - Gallargues-Le-Montueux - FRANCE

2

**BLUE WAYUP 3**

**NAUSICAA**

MEDICAL

3

Poids maximum  
Maximum weight  
Maximale tragfähigkeit  
Peso maximo  
Peso massimo

**150 KG**

CE

4

**BATTERY MAINTENANCE**

**NAUSICAA**

MEDICAL

**A POORLY MAINTAINED BATTERY DETERIORATES**

- Regularly charge the battery at the maximum.
- Batteries must be used and stored at temperatures between 5°C and 40°C.
- The charger should be checked if the battery stops working.
- Making small incomplete charges shortens the battery life.

**LEAD BATTERY**

- Full charge twice a week when used at home.
- Battery lifetime depends on its correct loading.

**LITHIUM BATTERY**

- Can be recharged as often as desired.
- If the charge level indicator remains off, recharge the battery.

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5

Instruction manual / Stand up lift : BLUE WAYUP 3

6

7

**QR CODE**

Instruction  
manuel

Instruction manual / Stand up lift : BLUE WAYUP 3

# Preventive Maintenance & Safety Checks

Type of stand up lift : BLUE WAYUP 3

Chassis serial number :

Maintenance must be carried out by trained personnel.

Check safety points (frequency: depending on use, at least once a year): this check is carried out visually; any sign of damage must be replaced.

ENGLISH

|    |                                                         | COMPLIANT | NO COMPLIANT | GREASING | DATE OF CHANGE |
|----|---------------------------------------------------------|-----------|--------------|----------|----------------|
| 1  | Bolted cylinder clevis mounting                         |           |              |          |                |
| 2  | Cylinder clevis mounting (bolted)                       |           |              |          |                |
| 3  | Bolt-on mast / lift arm assembly                        |           |              |          |                |
| 4  | Fixing wheel pivots (screwed, and bolted in some cases) |           |              |          |                |
| 5  | Fixing the baseplate to the mast (knurled knob)         |           |              |          |                |
| 6  | Knee rest attachment (knob)                             |           |              |          |                |
| 7  | Structural condition (welds, joints, oxidation)         |           |              |          |                |
| 8  | Motorized wheel mounting                                |           |              |          |                |
| 9  | Etat des cables et des connectiques                     |           |              |          |                |
| 10 | Check labels for presence and legibility                |           |              |          |                |
| 11 | Check platform travel and fasteners                     |           |              |          |                |

\*use a 3 in 1 product or similar

Check electrical functions (frequency: depending on use, at least once a year) :

this check is carried out with the Verticalisator under load.

|   |                                              | COMPLIANT | NO COMPLIANT | DATE OF CHANGE |
|---|----------------------------------------------|-----------|--------------|----------------|
| A | Lifting cylinder operation (up/down)         |           |              |                |
| B | Emergency stop operation                     |           |              |                |
| C | Battery status (charge retention)            |           |              |                |
| D | Remote control status                        |           |              |                |
| E | Motorized wheel operation (forward/backward) |           |              |                |
| F | Brake operation                              |           |              |                |
| G | Absence of suspicious noise                  |           |              |                |

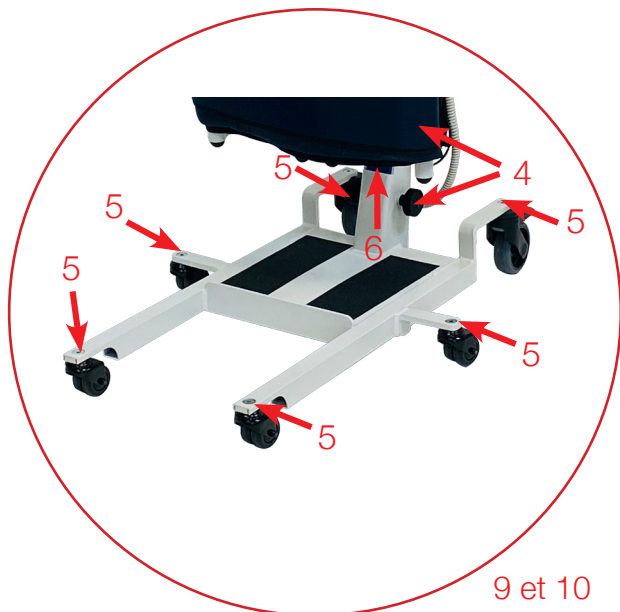
|                             |                             |                       |
|-----------------------------|-----------------------------|-----------------------|
| Inspections carried out on: | Inspections carried out by: | Next inspection date: |
|-----------------------------|-----------------------------|-----------------------|

## Upper section common to all 3 models


# Preventive maintenance and safety checks

## Foot fixe

## ECP mechanical or electrical
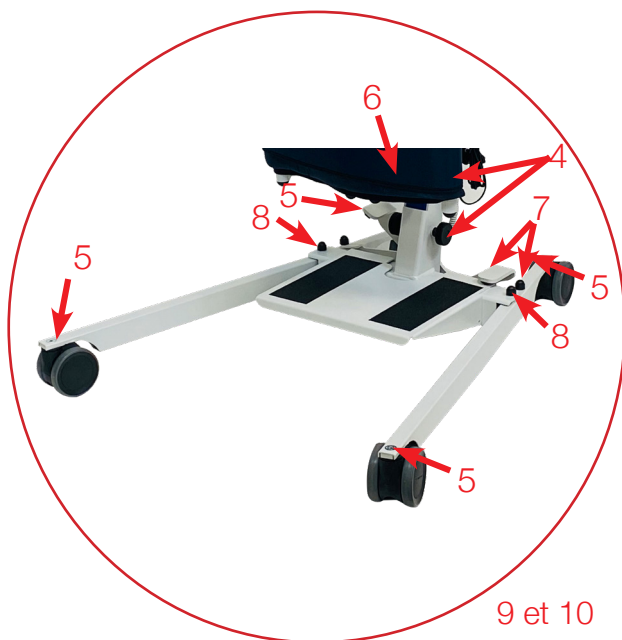
# Preventive maintenance and safety checks

## GREASING BLUE WAYUP 3

- 1- Cylinder / mast assembly
- 2- Mast / Lift arm assembly
- 3- Wheel pivot
- 4- Knee support assembly
- 5- Pedal or ECP cylinder assembly
- 6- Base / mast assembly
- 7- Pull strap assembly

Periodic greasing: use a “3 in 1” product or similar

### Upper section common to all 3 models

### Fixed foot

### Mechanical or electrical ECP

# Safety instruction

1. Use the sit-to-stand lift only for its intended purpose, in compliance with the legislation in force for medical devices, the legal requirements concerning them, the regulations for work protection and accident prevention.
2. Note that the lift is a medical device and that the user must therefore comply with the regulations governing the use of medical devices.
3. The electrical installation requirements for the room or area where the Verticalizer is used must comply with the current state of the art.
4. Do not use the lift unless you have been trained in its use and have full knowledge of its contents.
5. Read the entire instruction manual before use to avoid damage caused by improper handling or exposure to hazards. The instruction guide contains important information and instructions for using the lift.
6. Before use, check that the lift and its accessories are in perfect working order and condition.
7. Before using the sit-to-stand lift with other medical or non-medical devices, check that the combination of these products is permitted and that they can be used together safely.
8. Assembly, commissioning, maintenance and repair of the sit-to-stand lift may only be carried out by qualified specialists.
9. It is the responsibility of the user or operator to ensure (by means of suitable measures and instructions) that mechanical stress on the loading cord (by bending, pulling, shearing, crushing) is excluded during loading or cleaning of the workpiece. This also applies to the electrical cables of other equipment used with the Verticaliser.
10. Observe the switch-on time and the maximum permissible load. These values must not be exceeded, otherwise safe operation will no longer be ensured.
11. Do not expose the Verticaliser to direct sunlight, heat or moisture.
12. Ensure that no moisture enters the electrical system.
13. Avoid applying mechanical stress to the electrical cords used. Pulling, bending or crushing electrical cords may damage them.
14. Recharge batteries in a well-ventilated area.
15. Electromagnetic or other interference between the lift and other equipment cannot be ruled out. If there is a risk of such interference, the source of the interference must be removed or the lift must not be used.
16. Interference caused by the use of portable communications equipment cannot be completely ruled out. A safety distance of at least 3.3 m must therefore be maintained to ensure safe operation of the lift.
17. Do not allow children to stand unsupervised near the lift.
18. The lift should not be used if any abnormal noises, damage or other malfunctions occur. In this case, do not connect the lift to the charger, but inform NAUSICAA Médical.
19. If the lift is damaged or defective, it must not be used or connected to the mains supply. Inform your dealer and instruct him to rectify the defect or fault.
20. Any serious incident arising in connection with the device should be reported to NAUSICAA MEDICAL and to the competent authority of the Member State in which the user and/or patient is established.

# Technical specifications and dimensions

- Steel frames
  - Laser-cut and bent sheet metal, tubes and sections
  - Baked epoxy paint
  - Ball-bearing wheels
  - Total weight compact version: 28 kg
  - Total weight of mechanical ECP version: 32 kg
  - Total weight of electric ECP version: 35 kg
  - Weight of upper part: 16.40 kg (with one axle and two knobs)
  - Weight of base plate, compact version: 11.60 kg (with other axis and two knobs)
  - Weight of ECP mecha version base: 15.60 kg (with other axis and two knobs)
- ECP elec version base weight: 14.60 kg (with other axis and two knobs)

Maximum load: 150 kg

Product life: 8 years\* (excluding electrical part)

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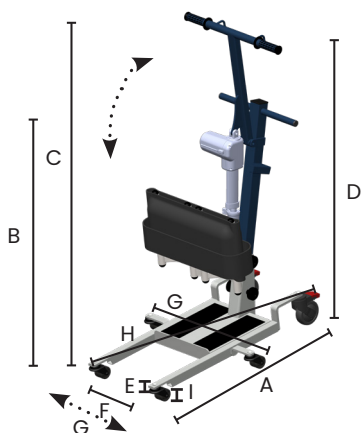
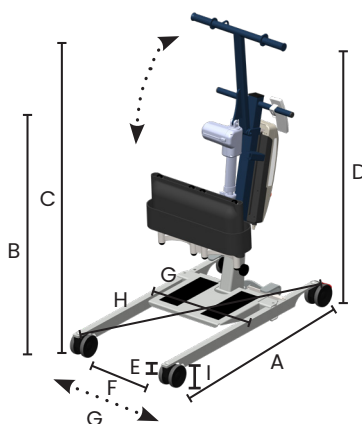
Mechanical and electrical ECP model

### Dimensions

- A.** Overall length: 1000 mm
- B.** Minimum useful height: 840 mm
- C.** Max. usable height: 1170 mm
- D.** Total height: 947 mm
- E.** Frame height: 115 mm
- F.** Min. baseplate width: 570 mm
- G.** Max. baseplate width :  
- mechanical spacing: 810 mm  
- electrical spacing: 800 mm
- H.** Turning diameter: 1120 mm
- I.** Ground clearance: 40 mm

# Spare parts

---

**For all spare parts requests, please  
contact our Customer Service De-  
partment:**

**Verticalizers & Lifts**

**Telephone: 04 66 71 71 80**

**Fax: 04 66 71 71 81**

**Mail : [sav@nausicaa-medical.com](mailto:sav@nausicaa-medical.com)**

# Cleaning and maintenance

## BEFORE HANDLING:

- Unplug power cord.
- Check that all electrical components are connected.
- Clean the electrical enclosures of the cylinders and wire controls, if these have been splashed with body fluids, especially urine.

## DAILY MAINTENANCE / BETWEEN PATIENTS :

- Clean with surface cleaner.
- Clean electrical jacks and wired controls if they have been splashed with body fluids, especially urine.
- Clean and disinfect all parts in direct contact with the patient (handles, sub-patellar support foam, etc.).

## COMPLETE CLEANING AND DISINFECTION :

- Clean the entire chassis with a pre-disinfectant detergent.
- Clean the motorization with a cloth slightly moistened with a pre-disinfectant detergent solution.
- Never clean the unit, especially the electrical system, with a high-pressure cleaner, water hose or similar.
- Disinfect the unit (e.g. by spraying with a surface disinfectant or misting with a disinfectant solution), observing the contact times of the product used.

## MAINTENANCE AFTER CLEANING AND DISINFECTION :

- Check the unit for visible damage.
- Check that no parts are missing.
- Check that the wheels are in good working order and that no material is interfering with their running (hair, carpet ends, etc.).
- Check that all controls are working properly, and that the remote control and actuator are connected to the battery pack.
- Check the integrity of the electrical cables (jack and remote control).

## CAUTION :

- **Detergents must be pH-neutral.**
- **Avoid abrasive products and solvents, as they may damage the surface of the unit.**

If you notice a fault, please contact our Customer Service department.:

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Mail : sav@nausicaa-medical.com

# Motorization technical data

- Battery pack
- Digital operating time display
- Emergency stop button
- Electrical safety lowering
- Independent wall charger (optional)
- 2-function/4-function remote control
- Complies with EN 60601-1 standards
- 24V= low-voltage DC motor
- Power 24 V/120 VA
- Maximum thrust force: 6,000 N
- Stroke: 11 cm
- Electronic overload protection
- Protection class: II
- Typical protection :
- Remote control: IP65
- Control box: IP54
- Battery pack: IP55
- Contactor limit switches
- ABS housing
- Remote control with coiled cable, low voltage 5V = overload protection by thermo-switches
- Overload protection by thermo-switches
- Cylinder noise level: 60 dB at 1 meter

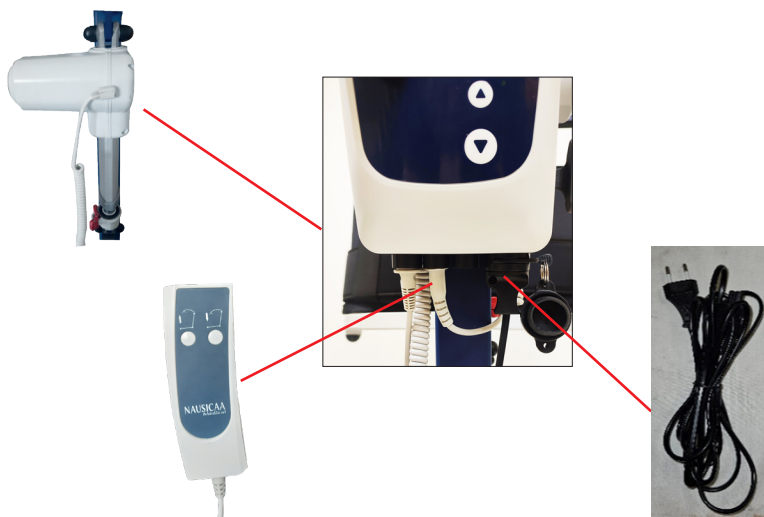
Appareil de  
classe II

ENGLISH



## Connection diagram

# Motorization technical data

## Control box display :

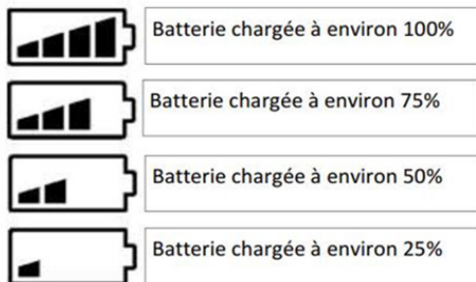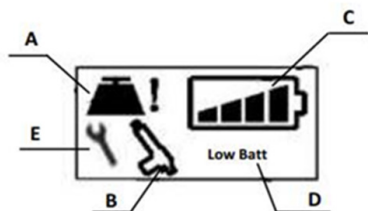The control unit is equipped with a display to communicate various information to user.

**A-** Overload indicator: The load to be lifted is too heavy and could damage the lift and endanger users. It is necessary to relieve the load on the lift.

**B-** Abnormal operating time indicator: The lift has been used for too long compared to its inactive time. This is a safety feature designed to protect the integrity of the lift. The expected operating ratio is 10% operating time to 90% idle time. Allow the lift to rest before attempting to use it again.

**C-** Battery level indicator.

**D-** Low battery indicator. Battery may be damaged and must be recharged before further use. An audible signal warns the user when the battery reaches 25% of its charge.

**E-** Cylinder maintenance or replacement indicator. The unit has been in use for a total of 600,000 seconds, i.e. approximately 10,000 cycles. A general overhaul of the entire unit is necessary.

# Troubleshooting guide

| Symptoms                                                                                              | Causes                                                                                                                                                                                                                                                                                                   | Solutions                                                                                                                                                                                                                                                                                     |
|-------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Uncharged device does not always descend.                                                             | Our aircraft need to have weight exerted on them in order to descend.                                                                                                                                                                                                                                    | Press down on the lifting arm while pressing the down button on the remote control.                                                                                                                                                                                                           |
| The moving parts of the unit are hard, the unit is hard to handle.                                    | This is due to a lack lubrication of moving parts.                                                                                                                                                                                                                                                       | Lubrifier les parties mobiles.                                                                                                                                                                                                                                                                |
| Jack does not work, but a "click" can be heard in the control box when the remote control is pressed. | <ol style="list-style-type: none"> <li>1. The batteries are discharged.</li> <li>2. Cylinder cable not connected.</li> <li>3. Cylinder cable damaged.</li> <li>4. Cylinder or control box damaged.</li> </ol>                                                                                            | <ol style="list-style-type: none"> <li>1. Load the unit.</li> <li>2. Connect cylinder cable.</li> <li>3. Have cylinder repaired.</li> <li>4. Have electrical assembly serviced.</li> </ol>                                                                                                    |
| Jack does not work, but no "click" is heard in the control box when the remote control is pressed.    | <ol style="list-style-type: none"> <li>1. Emergency stop button pressed.</li> <li>2. Battery discharged.</li> <li>3. Remote control cable not connected.</li> <li>4. Remote control cable damaged.</li> <li>5. Complete electrical system damaged.</li> <li>6. Battery not properly clipped..</li> </ol> | <ol style="list-style-type: none"> <li>1. Turn knob to unlock.</li> <li>2. Check charger status then charge the battery.</li> <li>3. Plug in remote control.</li> <li>4. Change remote control.</li> <li>5. Have electrical assembly serviced.</li> <li>6. Clip battery correctly.</li> </ol> |
| Cylinder stops.                                                                                       | <ol style="list-style-type: none"> <li>1. Battery discharged (buzzer sounds).</li> <li>2. Device has been used too long.</li> <li>3. Cylinder incorrectly connected</li> </ol>                                                                                                                           | <ol style="list-style-type: none"> <li>1. Check charger condition then charge the battery.</li> <li>2. Allow the unit to stand for a moment (to protect the cylinder life).</li> <li>3. Connect cylinder</li> </ol>                                                                           |

# Guarentee

Article 1: NAUSICAA Médical S.A.S. guarantees this device against all manufacturing and assembly defects in its mechanical and electrical components, and only for devices used under the conditions stipulated by NAUSICAA Médical S.A.S..

The warranty covers both mechanical and electrical parts.

This warranty, the conditions of which are defined below, is valid for 5 years, except for batteries (lithium battery: 2-year warranty, lead battery: 1-year warranty).

- Article 2 : The warranty entitles the purchaser to free labor and replacement of any parts found to be defective.

- Article 3 : The outward carriage of the device, as well as all related costs, are at the reseller's expense. Goods travel at the reseller's risk and responsibility.

Under warranty: NAUSICAA Médical S.A.S. will bear the cost of return shipment.

Out of warranty: return shipping costs are at the retailer's expense, whether or not he accepts the repair estimate.

- Article 4: The warranty does not apply if claims are the result of :

- accident, misuse or negligence on the part of the purchaser.

- transport of the device without adequate protection.

- modifications or alterations not approved by NAUSICAA Médical S.A.S.

- the impact of external factors (natural disasters, fire, impact, humidity, flooding, lightning, etc.).

- installation and/or use that does not comply with technical and safety standards, in the event that the device is operated in a country other than the country of purchase; and/or if the electrical supply is not adapted to the device's operating voltage.

- failure to carry out routine maintenance.

- Article 5: The retailer may not invoke the benefit of the warranty:

- if the device's serial number has been removed, modified or rendered illegible.

- if the device under warranty has been modified without the approval of NAUSICAA Médical S.A.S.

- Article 6: No loan of equipment will be made while defective equipment is being repaired.

- Article 7: All warranty claims must be made through the dealer. If this is not possible, the purchaser may send the equipment directly to NAUSICAA Médical S.A.S. In this case, the purchaser must indicate the retailer's contact details and a copy of the purchase invoice in a letter enclosed with the equipment.

- Article 8: Spare parts under warranty will only be sent after consultation with NAUSICAA Médical S.A.S.'s After Sales Department.

Please note that defective spare parts must be returned to NAUSICAA Médical S.A.S.'s After Sales Department, failing which they will be invoiced 1 month after NAUSICAA Médical S.A.S. has sent the parts.

- Article 9: Defective parts changed under or outside the warranty will be guaranteed for 6 months from the date of repair or shipment of spare parts.

- Article 10: No retailer may unilaterally modify the terms of the present warranty.

# NAUSICAA

## MEDICAL

ENGLISH

Head Office Sales Department

Phone : 04 66 51 50 80

Fax : 04 66 51 50 47

Mail : [contact@nausicaa-medical.com](mailto:contact@nausicaa-medical.com)

[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

Your contact :

Manufactured by NAUSICAA Médical S.A.S.

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- modificaciones o alteraciones no aprobadas por NAUSICAA Médica S.A.S.
- si el aparato en garantía ha sido modificado sin la aprobación de NAUSICAA Médica S.A.S.
Fabricado por NAUSICAA Médical S.A.S.