



www.nausicaa-medical.com

This medical device is intended for patient holding & positioning. It is not a restraint system.
Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

ECO SUPPORT BELT

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
FOR ALL CHAIRS



TO BE USED ONLY FOR THE
INDICATED USE :
TO PREVENT FROM FALLING

Maximal strap length : + 164 cm
Belt height : + 11 cm
Buckle : 50 mm

Simple and one size fits all, it is designed for different chair uses.
Maximum useful area: 164 cm
Without size constraints



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.
This medical device must be implemented or trained by a health professional.



CCMED2HP-PF

v 05.02/23

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



ECO SUPPORT BELT

HOSPITAL RANGE
Combination of polyurethan foam
and on both faces
Jersey/PU coating
100 % polyester bias
100 % polyester strap
Nylon buckle

TO BE USED ONLY FOR THE INDICATED USE :
TO PREVENT FROM FALLING

- Place the belt around the patient's abdomen (at hip level) and close the loop on the back of the chair.