



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
www.nausicaa-medical.com

This medical device is intended for patient holding and positioning. It is not a restraint system. Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

ECO YZA BELT



An abdominal belt, plus a single-sized pelvic module allow you to switch from an abdominal support to a pelvic support according to changing needs without size constraints.



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative. This medical device must be implemented or trained by a health professional.



CECO3-PF

v 05.02/23

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



ECO YZA BELT

HOSPITAL RANGE

Polyester lined polyester jersey lined
polyurethane foam

Bias 91% polyamide - 9% elastane

100% polyester webbing

PVC Shoulder Pad

Nylon buckle

The ECO Belt comes in 2 modular variants:

1-ECO Abdominal Belt (CECO-CA)

a - Position the belt at the waist of the person.

b - Pass the straps behind the back of the chair.

c - Close the nylon loop, then adjust the length of the straps, being careful not to compress the patient's abdomen.

2 -The Pelvic ECO Belt (CECO-P) consists of the ECO Abdominal Belt to which is added a Pelvic Support Module.

a - Position the pelvic support on the seat of the chair, and fix the 25 mm strap to the back of the chair with the minimum of play.

b - Sit the patient on the chair and on the strap.

c - Pass the straps behind the back of the chair.

d - Close the nylon loop, then adjust the length of the strap, being careful not to compress the patient's abdomen.

e - Close the 50 mm buckle of the pelvic support by adjusting its length.