

## PRODUCT DATA SHEET NAUSIFLOW 2S PUMP



### Technical features :

- Mechanical pump
- Automatic pressure maintenance
- Mechanical comfort adjustment to suit patient (positioning, weight and/or sensation)
- Visual and audible low-pressure alarm
- Normal pressure indicator
- Can be combined with 13 or 20 cm air mattresses up to 90 cm wide

### Control panel description :

- 2 operating modes :
  - Dynamic (default mode) : alternating pressure
  - Static (palliative care) good compromise between comfort and therapeutic benefits for patients.
- Comfort settings via rotary knob : in combination with palliative care button / soft / standard / firm / sitting
- Sitting adjustment before the patient is raised above 30°
- Low pressure indicator
- Normal pressure indicator
- Alarm indicator
- Audible alarm sleep button



### Indications :

Maximum user weight: **130 kg**  
Recommended for home use

#### Warning!

**In the event of a power failure, disconnect the mattress from the compressor and set the transport mode plug. Depending on duration, repeat clinical assessment of patient.**

#### Conditions of validity :

The data in this product data sheet is valid at the date it was drawn up. Any evolution of the product does not to be systematically updated if the fundamental characteristics of the product remain unchanged.