



www.nausicaa-medical.com

This medical device is intended for patient positioning. This is not a restraint system.

Before each use, it is important to check the presence of the label, the stitches of the straps, that they do not fray or break, that the device does not show any tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fasteners.

BUST SUPPORT BELT

FOR ALL CHAIRS



TO MAINTAIN THE CHEST IN
UPRIGHT POSITION

Maximum useful area: + 190 cm
Buckle: 40 mm

It provides reinforced comfort of the bust.

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
- Declined in 3 sizes S, M, L corresponding to the sizes of clothing.
- Maximum useful area: 190 cm



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative.



This medical device must be handled only by employees trained (by a healthcare professional) and qualified for its use (intended purpose, installation and environment of use, etc.). If the medical device is incorrectly installed, there is a risk of accident for the patient.



CMB2HP-PF



v 07.02/25

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



BUST SUPPORT BELT

HOSPITAL RANGE
JERSEY POLYESTER ENDUIT DE
POLYURETHANE DOUBLE DE MOUSSE
POLYURETHANE
BIAIS 100% POLYESTER
SANGLES 100% POLYESTER
BOUCLES NYLON

- Put on the breast support through the head.
- Pass the belt strap through the loop which is situated at the end of the support braces and grasp the backrest of the chair.
- Adjust length of straps depending on the needs.