

U SHAPE SLING WITH BACKREST


- Only available in comfort jersey
- Sizes S, M, L, XL.

The back section, extended by the rigid foam headrest and full whalebone, provides support for the spine during transfers and picking up from the floor. Its U-shape, combined with the net under the buttocks, makes it easier for the patient to put on and take off, and easier for the carer as well.

This very rigid sling prevents pressure or bending during transfers. The fastenings are taken up at the floating ribs to wrap the patient perfectly. It is particularly suitable for patients who have undergone arthrodesis. It can only be fitted to a patient in a seated position.



HOOK ADJUSTMENT

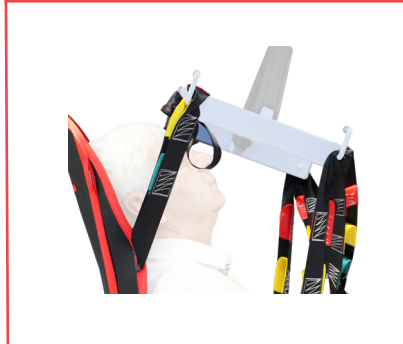

The multiple suspension system allows the patient's position to be varied. For optimum comfort in the seated position, we recommend adjusting the sling to an intermediate position (yellow setting) and as long as possible on the legs (black setting).

SLINGS POSITIONNING



CONFORT STYLE BIEN-ÊTRE
FABRICATION FRANÇAISE

REF.	SGLDOS-PF- (jersey comfort)
-------------	-----------------------------

