

QUICK ADJUST U SHAPE SLING

Compatible with transfer rails, ceiling rails and gantries.

- Available in comfort jersey and comfort mesh
- Sizes S, M, L, XL.



CONFORT STYLE BIEN-ÊTRE
FABRICATION FRANÇAISE

REF.	SST-PF (jersey comfort) SSTBA-PF (comfort mesh) SML (comfort jersey with integrated whale-bone headband) SMLBA-PF (comfort mesh with integrated with integrated boning)
OPTION	TAMOV-PF (removable headrest)



The U-shaped design prevents the slings from being placed under the patient's buttocks, making fitting and removal more comfortable for the patient and easier for the carer.

The foam leg slings reduce pressure on the thighs, and the two attachment points with two settings (particularly suitable for 2-point flails) allow quick combinations for seated, semi-recumbent and recumbent transfers.

HOOK ADJUSTMENT




The large buckle lets you adjust the angle without having to unhook the sling. The 2 lines allows you to move from a seated to a reclined position. For optimum comfort in the seated position, we recommend adjusting the sling as short as possible on the shoulders and as long as possible on the legs.

SLINGS POSITIONNING



Recommended choice

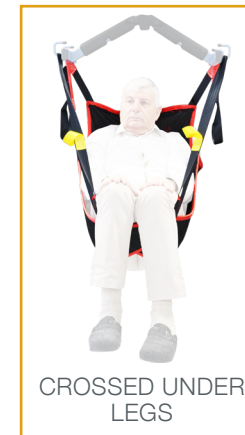

Second option

HEADREST (OPTION)

The addition of the removable headrest (TAMOV-PF) protects the neck when picking up from the ground, for example (only recommended for model ref.: SST).

