

# PREMIUM JERSEY TOILET SLING

The widened U-shape of this sling completely releases the patient's buttocks, making it easier to position the patient on the toilet, and makes the sling easier to put on and take off.

The legs pass from the outside to the inside. This sling is positioned with the arms on the outside.

**This type of sling is not recommended for patients with very low muscle tone.**

- Only available in jersey
- Sizes S, M, L, XL.

## HOOK ADJUSTMENT



The multiple hooks system allows the patient's position to be varied. For optimum comfort in the seated position, we recommend adjusting the sling to the intermediate position (yellow setting) and as long as possible on the legs (black setting).

## SLINGS POSITIONNING



STRAIGHT

Recommended choice



CROSSED BETWEEN LEGS

Recommended choice



CROSSED UNDER LEGS

Other option

## HEADREST (OPTION)

The addition of the removable, whalebone headrest (TABAL-PF) helps to protect when picking up on the ground.



CONFORT STYLE BIEN-ÊTRE  
FABRICATION FRANÇAISE



REF.	STO-PF-
OPTION	TABAL-PF (removable and underwired headrest)