

QUICK TRANSFER SLING

Easy to fit in a seat corset et Removable and underwired headrest.

Its enlarged U shape completely clears the patient's buttocks, making it easier to place on the toilet and making the sling easier to put on and take off. This sling has a wide headband and back foam. The sling is positioned with the arms outwards.

The sling runs from the inside to the outside. It prevents constriction of the hips when used uncrossed.

This type of sling is not recommended for patients with very low muscle tone.

- Only available in jersey comfort
- Sizes S, M, L, XL.

HOOK ADJUSTMENT




The multiple hook system allows the patient's position to be varied. For optimum comfort in the seated position, we recommend adjusting the sling to an intermediate position (yellow setting) and as long as possible on the legs (black setting).

SLINGS POSITIONNING



STRAIGHT

Recommended choice

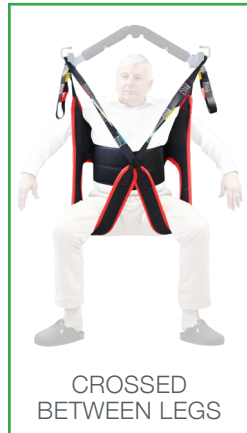

CROSSED BETWEEN LEGS

Recommended choice



CROSSED UNDER LEGS

Other option

HEADREST

The addition of the removable, whalebone headrest (TABAL-PF) protects the cervicals when picking up on the ground, for example



CONFORT STYLE BIEN-ÊTRE
FABRICATION FRANÇAISE



REF.	SATL-PF
OPTION	TABAL-PF (removable and underwired headrest)