

## PRODUCT DESCRIPTION

# PUMP NAUSIFLOW 100 AUTO QUATTRO



### Technical Features :

- Electronic management
- Automatic pressure maintenance depending on selected mode
- Comfort adjustment according to patient requirements (position, weight and/or comfort)
- Visual and audible alarm in the event of low pressure
- Suitable for 13 or 20 cm air mattresses - from 70 to 120 cm wide
- 30 minutes care mode

### Control Panel Description :

- 5 operating modes :
  - Dynamic (default mode): alternating pressure
  - Stimulation : rapid increase of pressure to generate a vasodilation reflex
  - Progressive: random cycle mode (to avoid continuity)
  - Static: good compromise between comfort and therapeutic cares
  - Care: provides a firm, stable surface. Ideal for nursing and/or repositioning
- 3 comfort settings: Soft / Standard / Firm allows adjustment according to the morphology and comfort felt by the patient
- 1 seat adjustment. Prevents the cell from being crushed. To be carried out before the patient is raised more than 30°!
- Control panel lock/unlock button
- Alarm indicator
- Audible alarm standby button.



### Indications :

- Maximum user weight : 150 kg
- Classe I device
- Suitable for home care and community.

### Warning :

**In the event of a power cut or failure, leave the mattress connected to the compressor and, depending on the duration, repeat the clinical assessment of the patient.**

Terms of validity :

The data in this product data sheet is valid at the issuing date. Any product upgrade does not systematically requires a product data sheet change if the fundamental characteristics of the product remain unchanged.