

PRODUCT DATASHEET

BLUE WAYUP 3 STAND UP LIFT

COMPACT 2 PARTS

DESCRIPTION :




- Steel frames
 - Laser-cut and bent sheet metal, tubes and sections
 - Baked epoxy paint
 - Ball-bearing wheels
 - Total weight of compact version: 28 kg
 - Weight of top section: 16.40 kg (with one axle and two wheels)
 - Weight of base: 11.60 kg (with other axle and two wheels)
 - Maximum load: 150 kg
 - Product life: 8 years* (excluding electrical part) * Subject to regular maintenance
 - Lithium battery: 2 year warranty
 - Lead battery: 1 year guarantee
- Medical devices (Regulation (EU) 2017/745)
 - Electromagnetic compatibility
 - Compliance with the following standards
 - NF EN 60601-1 : A1/2013
 - NF EN 60601-1-2 : 2015
 - NF EN ISO 10535 : 2021
 - 24V low-voltage DC motor
 - Power 24 V/120 VA
 - Maximum thrust force: 6,000 N
 - Electronic overload protection
 - Protection class: II
 - Type of protection: IP X4
 - Contactor limit switches
 - ABS housing
 - Sound level : 60 DB at 1 metre

MOTOR TECHNICAL FEATURES :

- Control box with built-in battery
- Battery life indicator light
- Power cord
- Emergency stop button
- 2-function remote control IPX4 C protection
- Emergency lowering: Electric & Manual
- Mains cable connection for battery charging

DIMENSIONS :


- A.** Total length: 91.5 cm
- B.** Minimum useful height: 77 cm
- C.** Maximum useful height: 116 cm
- D.** Total height: 94.5 cm
- E.** Frame height: 8 cm
- F.** Minimum base width: 39 cm
- G.** Maximum base width: 60.5 cm
- H.** Turning diameter: 89 cm

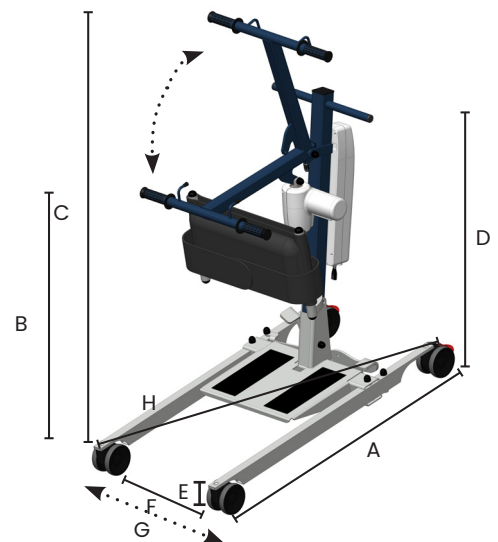

STORAGE :

To save space in your warehouses, we can make your appliance more compact for storage. Please follow the instructions below :

- Use the remote control to raise the arm of your lift by about 5 cm
- Loosen the retaining screw and lower the mast of your lift toward the ground.

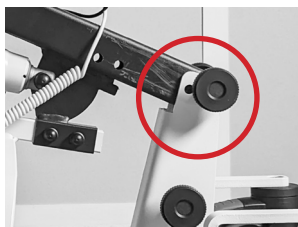

- Insert a sheet of paper or cardboard between the footrest and the shin support so that the non-slip surface of the footrest does not damage the fabric of the shin support.
- Pass the Velcro strap of your shin support under the footrest so that the device remains closed when stored upright



Terms of validity :

The data in this product data sheet is valid at the issuing date. Any product upgrade does not systematically requires a product data sheet change if the fundamental characteristics of the product remain unchanged.