



[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

This device is intended lifts and ceiling-mounted lifts for transferring a person to a sitting position.

It is compatible with lifts equipped with a smooth, open or closed hook system, being able to receive a 40 mm wide sling without impacting the strength of the sling on beams ranging from 2 to 4 points, which excludes hooking with a metal plate with holes.

## U-SHAPED SLINGS



Compatible with  
boned headrest  
(TAMOV).



- CE certified and manufactured in France, the Comfort Sling is recommended for bed to chair to bed transfers and for “lifting from the floor” transfers in a lying position.
- Its U-shape avoids the placement of the sling under the patient’s buttocks, making the installation and removal comfortable for the patient and accommodating for the caregiver.
- The leggings comfort foam reduces the shearing effect and the positioning of hooks at the shoulders level, traversing the entire slings dorsal part, avoids the phenomenon of compression during transfer.



Before each use, it is important to check: the presence of the label, the stitching of the straps, that they do not fray or come undone, that the device has no tears, snags, holes or signs of fraying (fabric and attachment straps), that the buckles are not cracked or have weak attachments.



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative. This medical device must be implemented or trained by a health professional.



SUCBAS-PF - SUCBAI-PF

v 05.02/23

### Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.




## U-SHAPED SLINGS

### SIZES & DIMENSIONS (in cm)

	A	B	C
S	59	53	75
M	65	57	83
L	70	61	92
XL	75	65	102

Composition  
100 % polyester fabric  
40 mm polyester strap - 1450 DaN  
Foam with polyester coating  
50 kg/m<sup>3</sup> PE foam  
40 mm polyester bias

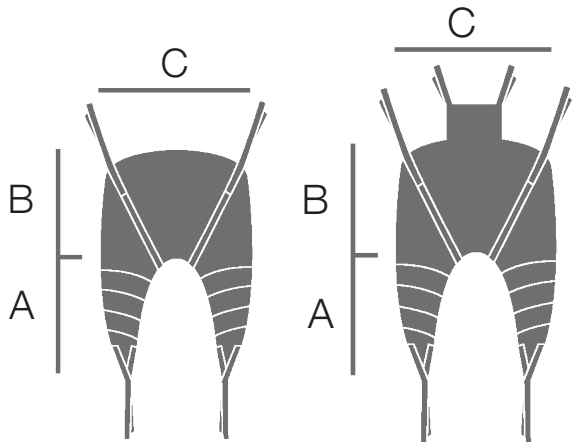

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This type of sling has a 4 points hanging system to the spreader bar :

- 1 at each end of the thigh support.

Position the lower part of the sling at the coccyx so that the loops of the hanging straps are on both sides of the patient. Slightly raise the right leg of the patient, pass the right thigh support under the right thigh. Do the same for the left leg. Both hanging straps are positioned between the legs of the patient. Cross these hanging straps. Hang both hanging straps of the back and hanging straps of the thigh supports to the spreader bar.

- 2 on both sides of the back

### ADJUSTMENTS :

The different loops on the hanging straps allow various sitting positions.

### CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.