



www.nausicaa-medical.com

This medical device is intended for patient positioning. This is not a restraint system.
Before each use, it is important to check: the presence of the label, that the device does not show any tears, snags, holes or signs of fraying and the good condition of the hook and loop.

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
FOOT SUPPORT



TO USE ONLY FOR THE INDICATED USE :
KEEPING YOUR FOOT ON
THE WHEELCHAIR PALLET

Description

Keeps the foot on the pallet of the chair (with self-gripping attachment under and on the sides of the pallet).



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative. This medical device must be implemented or trained by a health professional.



LMDP-HP-PF

v 05.02/23

Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



FOOT SUPPORT

HOSPITAL RANGE

Polyurethane-coated jersey lined with polyurethane foam

Bias 100% polyester

Self-gripping 100% polyester

PU foam 100 kg / m³

- Attach each foot support to each wheelchair paddle with the self-gripping sole of the foot support.
- Place each foot of the patient in a footrest.
- Close the footrest on the patient's foot with the self-gripping device.

