



[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

This medical device is intended for patient holding and positioning. It is not a restraint system.  
Before each use, it is important to check: the presence of the label, the stitches of the straps: that they do not fray or break, that the device does not show tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

## SIMPLE SLIM BELT

FOR ALL CHAIRS

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


TO BE USED ONLY FOR THE  
INDICATED USE :  
TO PREVENT FROM FALLING

Maximal strap length : + 230 cm  
Belt height : + 11 cm  
Buckle : 50 mm

This simple and robust system is common to all SLIM belts.

- Available in 4 sizes S, M, L, XL
- Maximum useful area: 230 cm
- Waist (in cm)



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative. This medical device must be implemented or trained by a health professional.



CCS2HP-PF

v 05.02/23

### Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



## SIMPLE SLIM BELT

HOSPITAL RANGE  
Coated polyester jersey  
doubled polyurethane  
polyurethane foam  
Bias 100% polyester  
100% polyester straps  
Nylon buckles

- Position the belt at the waist of the person.
- Close the belt using the Velcro closure.
- Pass the straps behind the back of the chair.
- Close the nylon buckle and adjust the length of the straps, making sure not to compress the patient's abdomen.