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




[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

Before each use, it is important to check the presence of the label, the stitches of the straps, that they do not fray or break, that the device does not show any tears, snags, holes or signs of fraying (hanging fabric and straps)

## SLIDE SHEETS WITH HANDLES



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative. This medical device must be implemented or trained by a health professional.



DTR-PF

v 05.02/23



# SLIDE SHEETS WITH HANDLES

100% polyamide fabric:  
- Silicone coating (top)  
- Polyurethane coating (bottom)

## DESCRIPTION

- For the handling of a patient, the Slide Sheet is a device that facilitates lateral and vertical translation as well as the raising of a lying patient in a bed.
- The sliding effect facilitates the work of carer, promotes the prevention of pressure ulcers and improves patient comfort.
- The Slide Sheet is composed a of synthetic fabric polyamide tube in polyurethane induction, the two layers of fabric in contact slide on one another.
- Silicone Treatment on Patient Side.

## USES

