



[www.nausicaa-medical.com](http://www.nausicaa-medical.com)

This medical device is intended for patient support and positioning. It is not a restraint system. Before each use, it is important to check the presence of the label, the stitches of the straps, that they do not fray or break, that the device does not show any tears, snags, holes or signs of fraying (fabric and straps), that the buckles are not cracked or have no weak fastening.

## SECURITY MITTS



integrated  
aluminium  
stiffener

USES :  
COMFORTABLE, THEY PREVENT  
THE RESTLESS PATIENT FROM  
TEARING HIS CARE DEVICES AS  
WELL AS HIS INCONTINENCE PRO-  
TECTIONS.

### Description

The breathable net part on the top increases comfort and the integrated aluminium stiffener prevents the hand flexion.



The use of the device requires the presence and vigilance of a third person. When used by an agitated or disoriented person, monitoring becomes imperative. This medical device must be implemented or trained by a health professional.



LMMFL-HP-PF

v 05.02/23

### Important Recommendations :

- In order to get a maximum efficiency of these sort of product, it is necessary :
- to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



## SECURITY MITTS

### HOSPITAL RANGE

Combination of polyurethan foam  
and on both faces jersey/PU coating

100% polyester net

100% polyester bias

100% polyester strap

Nylon buckle

Aluminium stiffener

- Place the patient's hand in one of the security mitt, net part on the upper part of the hand.
- Close the nylon buckle, then adjust the tightening.
- Secure it with the velcro strap.